Löwenstein Prisma SMART/ SOFT Welcome to the Comfort Zone.







German-Made Device

Prisma SMART/SOFT continues the tradition of high-quality APAP/CPAP devices from Löwenstein Medical & offers the new option of classic or dynamic pressure regulation for personalised respiratory therapy.*

The familiar pressure relief "softPAP" can be activated at any time. Classic, simple user navigation with

easy-to-read display & state-ofthe-art technology incorporating FOT technology.

*This applies to the Auto devices.

The Prisma SMART/SOFT Therapy Set comes complete with:

- Device
- Humidifier tub (optional) & cover plate
- Carry Bag
- Power Supply
- User guide
- Breathing tube
- Heated breating tube (optional)
- x2 replacement filters
- SD Card



How can I find out more?

For visual content on how to set up the PrismaLINE & use the humidifier PrismaAQUA, scan the QR code or visit https://www.sante-group.com/sleep-device-video*library* for videos:











5-Year Warranty

The device itself comes with a 5-year warranty for manufacturing defects.* German-engineered & manufactured under stringent quality control, the

chances are very low of the device breaking down.

No fuss warranty for peace of mind!

*Assessment criteria applies



The tub size of 400mL is the largest currently available & mitigates the need to refill during the night.



The 4-component modular humidifier is easy to fill, disassemble, & clean.

The unique cylindrical heating element is a particularly strong component, built to last. It is not susceptible to mineral build-up or damage like some heating plates are, it also has sensors to turn off rather than overheat if the water level ever drops too low.



<u>Prisma SMART/</u> SOFT max device option with builtin connectivity

prisma20A Hodus P max P min Ø Nutzung 6:22 h	22.09. A740 7.192	19-06.11	Osturn				
Modus P max P min	APAP	19-06.11			Unters	with	
P max P min			19	SN 99	7927185		W 5.01.0004
P min	7109	A0 4.0		210	softSTAR	Flowerk	41/24
		NIFH		2			
A Manuary 6-021 h	4 hPa	selfs5M	Tman	15-min			
			Therapieda	uer 178:2	2 h		
Verteilung der Nutz	congedeser (h.T.	(ege			Tage		d Nation
			Gewählt		28	100 %	6.22 h
-	a Sh	71%	Cenulat		28	100 %	622 h
	8 4-5h	45	devon mit Bel	wathler	28	100 %	617h
	C 2-4h	14%			21	75 %	2:31 h
	10 × 3h	115					
AHI 11 /b			Periodische	Atmana	23		
All Verteik	ung (% Tage)		-	1.0	Schoanhan		0.5
			-	2.0	P50		71Pa
_	👗 = 5.m	14%	pAI	0.0	P90		7100
	B 5-10.h	25%	194	0.0			
۰ (ا	C 10-15/h	25%	AP8 1461.101	11/h			
~	D + 15 /h	36%					
Anteil mit hoher Le	ckage 0 %						
	ething (% Tage)		Leckage (Med	int i			25.1min
			Lockage (P95)				17,5.1/min
	A keine	82%					
	8 geringe	14%					
•	C deutiche	43					
	D state	0%					
	1 state	05					

4-WOCHEN-KALENDER	10.10.19 - 06.11.19	AUSWERTUNG	
Returnputters ()		Phone 121	Analog in Research of Part
Same Street all	Provide and	Genetari 28 Traje	i dilim.
The Local Division of Local Divisio of Local Division of Local Division of Local Div	Name - Date	A Designed and	Total States of American Street States
hattlathall	الإنداريان	Property a	ililata 🍖 🏣
Theophysikit (Logram)	Ban Dan Dans Ban D		Property of Street, or
	Lt. Inset of	and believed 11/24	
ALL ALL ADDRESS		Perioductor 2%	
Description 1/1	in lines in factories	Schearuben (1)	Contract of the second second
Suspection (17)	- Table of	" Hiterate 2.5	4
Contraction Contraction	No. Constant and a second second	-	14
		initial blue Linkage PL	 6
~	Due	and	10.00.00

Left & Above: Therapy calendar, reports & remote setting adjustments are possible for the clinician.

Right: Provides access to key statistics for the empowered patient. Includes Compliance, AHI, Leakage, Deep-Sleep Index.



Best Ireatment Outcomes.

The auto algorithm responds to events by automatically providing <u>continuous pressure adjustments</u> tailored exactly to your needs.

Based on feedback from experts & experienced PAP users, you can expect the following benefits when using Prisma devices:

- Auto treatment pressures are particularly efficient without unnecessary fluctuation. This puts less strain on lungs, allowing for more comfortable & natural breathing patterns & less leakage.
- There is minimal disruption as <u>the device</u> is whisper quiet.
- Users report a noticeably recuperative treatment outcome, i.e. more quality sleep, greater energy & even less required sleep hours. This ultimately leads to improved productivity in work, hobbies & livelihood!





Product Description	Product Code
Prisma SOFT CPAP Therapy Set with PrismaAQUA & Heated Tube	31650-1110, 29680, 29045
Prisma SOFT CPAP Therapy Set with PrismaAQUA	31650-1110, 29680
Prisma SMART APAP Therapy Set with PrismaAQUA & Heated Tube	31620-1110, 29680, 29045
Prisma SMART APAP Therapy Set with PrismaAQUA	31620-1110, 29680
Prisma SOFT max CPAP Therapy Set with PrismaAQUA & Heated Tube	31980-1110, 29680, 29045
Prisma SOFT max CPAP Therapy Set with PrismaAQUA	31980-1110, 29680
Prisma SMART max APAP Therapy Set with PrismaAQUA & Heated Tube	31960-1110, 29680, 29045
Prisma SMART max APAP Therapy Set with PrismaAQUA	31960-1110, 29680

Your treating doctor will prescribe the best suited device for your needs.



